RAMP® Total β-hCG Controls

REF

C4003-1

CONTROL

Level 1; 6 x 0.5 mL

1

INTENDED USE

The RAMP® Total β-hCG Controls are intended for *in vitro* diagnostic use in the quality control of the RAMP® Total β-hCG assay on the RAMP® Platform.

DESCRIPTION

The RAMP® Total β-hCG Controls are prepared from a buffered recombinant protein solution; preservatives have been added to maintain product integrity. The RAMP® Total β-hCG Controls are ready-to-use liquid controls requiring no reconstitution or frozen storage. The RAMP® Total β-hCG Controls are supplied at 2 levels, 1 and 2, with target values and ranges provided.

SAFETY PRECAUTIONS AND WARNINGS

For *in vitro* diagnostic use only. Exercise standard precautions for the handling of laboratory reagents in accordance with applicable laws and good laboratory practices. Avoid contact with skin; use gloves; in case of contact with skin, wash off with soap and plenty of water.

The device contains material of human or animal origin and should be handled as a potential biohazard. No known test method can offer complete assurance that products derived from human sources will not transmit infectious agents.

Dispose of containers and unused contents in accordance with Federal, State, and Local requirements.

STORAGE AND STABILITY

OPENED: Store refrigerated at +2°C to +8°C. RAMP® Total β-hCG Controls are stable for 30 days at +2°C to +8°C if kept capped in original container and free

from contamination. Only the required amount of product should be removed. After use, any residual product should NOT BE RETURNED to the original vial.

UNOPENED: The product is stable to the expiration date printed on the individual vials, when stored at +2°C to +8°C.

PREPARATION FOR USE

The RAMP® Total β-hCG Controls are supplied ready to use.

MATERIALS PROVIDED

Liquid Total β-hCG Control – Level 1 6 x 0.5 mL

MATERIALS REQUIRED BUT NOT PROVIDED

RAMP® Total β-hCG test kit
RAMP® instrument

ASSIGNED VALUES

Values are assigned from internal testing conducted at Response Biomedical Corp. The expected range of the mean is provided to aid a laboratory until it has established its own mean and SD for its methods.

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